

DERMACEUTIC

LABORATOIRE

Skintelligent Science

PROFESSIONAL PEELS

HD

AUTO

UVA

DERMACEUTIC





Skintelligent Science

Dermaceutic, the expert in aesthetic dermatology

Dermaceutic was founded in 2002 on the strong belief that **science is the skin's best ally**. We use the best scientific expertise to strengthen and support the skin's natural powers, for faster and more effective results.

Born from a collaboration between French dermatologists, Dermaceutic is now **the number one chemical peel brand in France**, and has brought its art of formulation to the cosmeceutical industry.

Today, we partner with more than 30,000 skin professionals in more than 80 countries around the world. By working closely with these specialists, we gain vital and ever-evolving insights into different skin types, phototypes and unique patient expectations.

We elevate skincare standards using the latest scientific technology and expertise, and by delivering exceptional results for every individual's skin. Whatever the skin concern or aesthetic procedure, our mission is to provide the right combinations of actives at the most effective concentrations.

We aim to maintain our position as **the expert in aesthetic dermatology** through our passion for scientific discoveries and our insatiable curiosity for skin science, our continued strategic partnerships with dedicated and passionate distributors, and through our strong relationships with a large network of experts in the scientific community.

This is Skintelligent Science.

Dermaceutic is the expert in aesthetic dermatology. We were born from a collaboration between French dermatologists and now partner with more than 30,000 skin professionals internationally. By working daily with these specialists, we gain vital and ever-evolving insights into different skin types, phototypes and unique patient expectations. Thanks to this expertise, Dermaceutic is the only French laboratory with a range of professional peels certified class IIa.

Very superficial peel p.4

MASK PEEL p.4

ACNE-PRONE SKIN



Superficial peels p.6

MILK PEEL p.6

MILD ACNE, KERATOSIS PILARIS, SUPERFICIAL PIGMENTED LESIONS



CRYSTAL PEEL p.8

MELASMA, SOLAR & SENILE LENTIGINES, POST-INFLAMMATORY PIGMENTATION, MILD ACNE, HYPERKERATOSIS



COSMO PEEL 12% & 15% p.10

SOLAR LENTIGINES, SUPERFICIAL PIGMENTED LESIONS, KERATOSIS PILARIS



Medium peels p.12

ACTIVA & MELA PEEL FORTE p.12

PIGMENT SPOTS



COSMO PEEL 18% & 20% p.14

SOLAR & SENILE LENTIGINES, PIGMENTED LESIONS, ELASTOSIS



TCA 30 p.16

SOLAR & SENILE LENTIGINES, ACTINIC KERATOSIS AND ACNE SCARS



COSMO PEEL FORTE p.18

ACNE SCARS, SENILE LENTIGINES, ACTINIC KERATOSIS



PEEL INTENSITY

Very superficial ●○○○○ Superficial ●●○○○ Medium ●●●○○ Deep ●●●●●

Very superficial peel

MASK PEEL

For the appearance of oily and acne-prone skin, dilated pores and excess sebum. Purifies the epidermis and tightens pores.



BENEFITS FOR THE SKIN*

- + Purifies the epidermis
- + Brightens **dull complexion**
- + Decreases **sebum production**
- + Tightens **pores**
- + Regulates **oily skin** and dilated pores

*Literature based evidences of AHA/BHA peels

ACTIVE INGREDIENTS

Glycolic and salicylic acids, bentonite
pH ≈ 2

POSSIBLE SIDE EFFECTS

Desquamation, erythema, oedema, redness, allergy and/or burning sensation may appear. If these symptoms persist for longer than 72 hours arrange a consultation with the patient.

CLINICALLY TESTED



IN-CLINIC PROTOCOL

4 times at 2 week intervals.

1. **Foamer 15** - Use to cleanse the face.
2. **Mask Peel** - Apply a pea-sized amount to the entire face and leave on for 2 minutes. Rinse with water and dry skin.
3. **Hyal Ceutic** - Apply to the face.

WHY CHOOSE MASK PEEL ?

- + Suitable for all skin types and phototypes
- + No social downtime (lunch peel)
- + Little to no skin desquamation
- + Safe & easy 10 min protocol
- + Can accompany antibiotic treatments

SYNERGY OF ACTIVE INGREDIENTS

Glycolic acid, an hydrophilic ingredient, to stimulate cell renewal and eliminate dead cells.

Salicylic acid, a lipophilic ingredient, for a keratolytic action on the skin (epidermis).

IDEAL FOR INTRODUCING PATIENTS TO PEELS

Very superficial peel with effective and visible results.

EXPERT TESTIMONIAL

“Mask Peel is a very easy product to use for skin with large pores, comedo and milia especially on the T-Zone. It visibly reduces the production of sebum and evens the skin texture.”

Dr. Tatjana Pavicic

Superficial peel

MILK PEEL

CE 0051

Intended for the treatment of mild acne, keratosis pilaris and superficial pigmented lesions.



Kit includes: Foamer 15 40 ml, Milk Peel 60 ml, brush and cup
For 24 applications

CE 0051 limited to Milk Peel bottle

Milk Peel is a class IIa medical device

BENEFITS FOR THE SKIN*

- + Reduces **acne lesions**
- + Eliminates superficial **pigmented lesions**
- + Improves the condition of keratosis pilaris through an **exfoliative effect**
- + Boosts **collagen** production
- + Improves appearance of **fine lines and wrinkles**

*Literature based evidences of AHA/BHA peels

ACTIVE INGREDIENTS

Milk Peel: Glycolic, lactic and salicylic acids
pH ≈ 1.9

Foamer 15: Glycolic acid, exfolone
pH ≈ 4.1

POSSIBLE SIDE EFFECTS

Desquamation, erythema, oedema, redness, allergy and/or burning sensation may appear. If these symptoms persist for longer than 72 hours arrange a consultation with the patient.

CLINICALLY TESTED



IN-CLINIC PROTOCOL

4 times at 2 week intervals.

1. **Foamer 15** - Use to cleanse the face.
2. **Milk Peel** - Apply to the entire face. Leave for 3 to 5 minutes. Rinse with water and dry skin.
3. **Hyal Ceutic** - Apply to the face.

WHY CHOOSE MILK PEEL ?

- + Suitable for all skin types and phototypes
- + No social downtime (lunch peel)
- + Little to no skin desquamation
- + Safe & easy 10 min protocol
- + Can be combined and enhances the efficacy of other aesthetic treatments

*In vivo efficacy study on 63 patients. European independent contract research organization.

PEEL INTENSITY



PROVEN RESULTS

92% of patients' **skin was more luminous.***

83% of patients said the **skin was more even and more radiant.***

100% of patients showed a **significant improvement in frown line roughness.***

EXPERT TESTIMONIAL

“ *Milk Peel is a safe, effective and progressive treatment which requires minimal recovery time. It is ideal to achieve optimal results, both when used alone and in combination with other treatments.* ”

Dr. Jules Marthan

Superficial peel

CRYSTAL PEEL

CE 0051

Intended for the treatment of melasma, solar & senile lentigines, post-inflammatory pigmentation, mild acne and hyperkeratosis.

BENEFITS FOR THE SKIN*

- + Treats **epidermal melasma**
- + Resolves **post-inflammatory hyperpigmentation**
- + Fades **pigment spots** and **post-acne pigmentation**
- + Improves **acne symptoms** (comedones, papules and pustules)
- + Improves **skin texture, mild photoaging** and **rough skin**
- + Reduces **pore size** and decreases **sebum**

*Literature based evidences of AHA/BHA peels

ACTIVE INGREDIENTS

Crystal Peel: Salicylic, mandelic and citric acids
pH ≈ 2.0

Foamer 15: Glycolic acid, enoxolone
pH ≈ 4.1

IN-CLINIC PROTOCOL

Up to 3 times at 3-week intervals.

- 1. Foamer 15** - Use to cleanse the face.
- 2. Crystal Peel** - Apply up to 3 layers to the entire face. Wait 1 minute between each layer. Leave the last layer up to 5 minutes. Rinse with water and dry skin.
- 3. Post-peel cream** - If needed, apply to the face the relevant Dermaceutic post-treatment cream, corresponding to the indication treated.

WHY CHOOSE CRYSTAL PEEL ?

- + Suitable for all skin types and phototypes
- + No social downtime (lunch peel)
- + Little to no skin desquamation
- + Safe & easy 10 min protocol
- + Can be combined and enhances the efficacy of other aesthetic treatments

POSSIBLE SIDE EFFECTS

Oedema, erythema, desquamation, persistent redness, itching, burning sensation or pigmentation disorder may appear after the treatment. If these symptoms persist for longer than 72 hours, arrange a consultation with the patient.

PEEL INTENSITY




SALICYLIC ACID




Lipid-soluble agent with powerful keratolytic and comedolytic properties that exfoliates and softens the stratum corneum, making it an effective peeling agent.

MANDELIC ACID

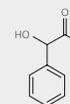

Large molecule offering low skin penetration to ensure a uniform epidermal effect, making it a safe superficial peeling agent ideal for sensitive skin.

CITRIC ACID

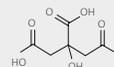

Small molecule able to disrupt the cohesion of corneocytes of skin barrier and thus induce keratolysis.



Kit includes: Foamer 15 40 ml, Crystal Peel 30 ml, cotton buds and cup
For 12 applications

CE 0051 limited to Crystal Peel bottle

Crystal Peel is a class IIa medical device

Superficial peels

COSMO PEEL

12 & 15%

CE 0051

Intended for the treatment of solar lentigines, superficial pigmented lesions and keratosis pilaris.



Kit includes: Foamer 15 40 ml, TCA 12 or TCA 15 30 ml, Cosmo Cream 40 ml, cup and cotton buds - For 18 applications

CE 0051 limited to TCA bottle (TCA 12 or TCA 15)

TCA 12 and TCA 15 are class IIa medical devices



BENEFITS FOR THE SKIN*

- + Eliminate **pigmented lesions**
- + Improve the condition of keratosis pilaris through an **exfoliative effect**
- + Boost **collagen** and **elastin production**
- + **Homogenize** and **revitalize** the skin
- + Improve appearance of **wrinkles** and **signs of premature aging**

*Literature based evidences of TCA peels

ACTIVE INGREDIENTS

Trichloroacetic acid 12%, 15%: pH ≈ 0.5

Cosmo Cream: Dextran sulfate, allantoin, hyaluronic acid, vitamin C, calendula extract
pH ≈ 4.8

Foamer 15: Glycolic acid, enoxolone
pH ≈ 4.1

POSSIBLE SIDE EFFECTS

Desquamation, erythema, frosting, oedema, redness, pigment rebound, allergy and/or burning sensation may appear. If these symptoms persist for longer than 72 hours arrange a consultation with the patient.

CLINICALLY TESTED



IN-CLINIC PROTOCOL

2 to 4 times at varying intervals, depending on the skin conditions to treat and the TCA concentration.

- 1. Foamer 15** - Use to cleanse the face.
- 2. TCA** - Apply to the face. Wait 1 to 2 minutes. Reapply additional layers if necessary. Do not rinse.
- 3. Cosmo Cream** - Massage into the face. Rinse between 4 and 8 hours later.

WHY CHOOSE COSMO PEEL ?

- + Controlled penetration thanks to buffered TCA
- + Wide range of concentrations:
 - + Epidermal to dermal stimulation for all phototypes
 - + Targeted to full face application

*In vivo efficacy study on 63 patients. European independent contract research organization.

PROVEN RESULTS

81% of patients showed a **significant improvement in wrinkles around the eyes.***

88% of patients have **less signs of skin aging.***

86% of patients showed a **significant and continuous improvement in the darkness of spots.***

EXPERT TESTIMONIAL

“ It is my favorite product to treat and prevent signs of aging. With its different concentrations, it gives me the flexibility to treat a wide range of skin phototypes and conditions while stimulating skin regeneration. ”

Dr. Jules Marthan



Medium peel system

ACTIVA PEEL FORTE & MELA PEEL FORTE

Indicated to reduce the appearance of pigment spots.



Activa Peel Forte* 12 ml is a registered professional aesthetic product AEMPS: 693-PE - For 5 applications

Mela Peel Forte* 15 ml is a registered professional aesthetic product AEMPS: 686-PE - For 5 applications

*Products sold separately

BENEFITS FOR THE SKIN*

- + Reduces the appearance of **melasma** and **pigment spots**
- + Evens **skin tone** and **complexion**
- + Improves **skin radiance** and **brightness**
- + Proven action on **all mechanisms of pigmentation**

*Literature based evidences of AHA/BHA peels

ACTIVE INGREDIENTS

Activa Peel Forte: Salicylic, azelaic, ferulic, citric and phytic acids
pH ≈ 0.9 - 1.3

Mela Peel Forte: Resorcinol, lactic, citric, kojic and phytic acids
pH ≈ 0.9

POSSIBLE SIDE EFFECTS

Side effects such as edema, erythema, peeling, persistent redness or pigmentation disorders may occur after treatment. If such symptoms persist for more than 72 hours, schedule a consultation with the patient.

CLINICALLY TESTED



IN-CLINIC PROTOCOL

2 times at a 1 month interval.

1. **Activa Peel Forte** - Pour 1.5 ml in a cup and use a cotton bud wrung out on the rim of the glass beforehand to evenly apply it on the face. Wait 1 to 2 minutes. Apply a second layer on the skin spots.
2. **Mela Peel Forte** - Apply 4 pumps and massage the skin. Reapply to stubborn spots. Massage the cream for 3 to 5 minutes.

WHY CHOOSE THIS SYSTEM ?

- + Recommended for phototypes I, II and III
- + Little to no social downtime
- + Minimal risks of side effects
- + Highly concentrated formulae for optimal efficacy
- + Long lasting results with on-going patient care

PROVEN RESULTS

83% of patients have a **more even skin tone.***

83% of patients showed a **decrease in age spots and hyperpigmentation.***

66% of patients showed a **significant improvement in skin clarity.***

EXPERT TESTIMONIAL

“It is a valuable adjunctive therapy for the treatment of various types of pigmentation. It is safe and well tolerated and is used to help improve pigmentation on my patients' skin.”

Dr. Mukta Sachdev

*In vivo study on 6 patients. Independant contract research organization.

Medium peels

COSMO PEEL

18 & 20%

CE 0051

Intended for the treatment of solar and senile lentiginos, pigmented lesions and elastosis.



Kit includes: Foamer 15 40 ml, TCA 18 or TCA 20 30 ml, Cosmo Cream 40 ml, cup and cotton buds - For 18 applications

CE 0051 limited to TCA bottle (TCA 18 or TCA 20)

TCA 18 and TCA 20 are class IIa medical devices

BENEFITS FOR THE SKIN*

- + Eliminate **pigmented lesions**
- + Reduce **photodamaged spots**
- + **Homogenize** and **revitalize** the skin
- + Boost **collagen** and **elastin production**
- + Improve appearance of **wrinkles** and **sagging skin**

*Literature based evidences of TCA peels

ACTIVE INGREDIENTS

Trichloroacetic acid 18%, 20%: pH ≈ 0.5

Cosmo Cream: Dextran sulfate, allantoin, hyaluronic acid, vitamin C, calendula extract pH ≈ 4.8

Foamer 15: Glycolic acid, enoxolone pH ≈ 4.1

POSSIBLE SIDE EFFECTS

Desquamation, erythema, frosting, oedema, redness, pigment rebound, allergy and/or burning sensation may appear. If these symptoms persist for longer than 72 hours arrange a consultation with the patient.

CLINICALLY TESTED



IN-CLINIC PROTOCOL

2 to 4 times at varying intervals, depending on the skin conditions to treat and the TCA concentration.

1. **Foamer 15** - Use to cleanse the face.
2. **TCA** - Apply to the face. Wait 1 to 2 minutes. Reapply additional layers if necessary. Do not rinse.
3. **Cosmo Cream** - Massage into the face. Rinse between 4 and 8 hours later.

WHY CHOOSE COSMO PEEL ?

- + Controlled penetration thanks to buffered TCA
- + Wide range of concentrations:
 - + Epidermal to dermal stimulation for all phototypes
 - + Targeted to full face application

*In vivo efficacy study on 63 patients. European independent contract research organization.

PEELS INTENSITY



PROVEN RESULTS

81% of patients showed a **significant improvement in wrinkles around the eyes.***

88% of patients have **less signs of skin aging.***

86% of patients showed a **significant and continuous improvement in the darkness of spots.***

EXPERT TESTIMONIAL

“ It is my favorite product to treat and prevent signs of aging. With its different concentrations, it gives me the flexibility to treat a wide range of skin phototypes and conditions while stimulating skin regeneration. ”


Dr. Jules Marthan

Medium peel

TCA 30

CE 0051

Intended for the treatment of solar and senile lentigines, actinic keratosis and acne scars.

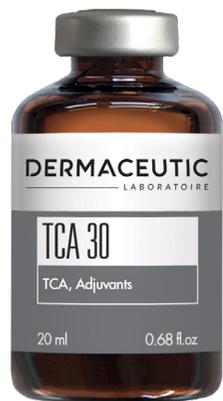

BENEFITS FOR THE SKIN*

- + Eliminates **pigmented lesions**
- + Eliminates **actinic keratosis** lesions
- + Promotes the removal of **acne scars** through targeted application
- + Boosts **collagen** and **elastin production**
- + Improves appearance of **deep wrinkles**

*Literature based evidences of TCA peels

ACTIVE INGREDIENTS

Trichloroacetic acid 30%: pH ≈ 0.5

POSSIBLE SIDE EFFECTS

Desquamation, erythema, frosting, oedema, redness, allergy and/or burning sensation may appear. If these symptoms persist for longer than 72 hours arrange a consultation with the patient.

IN-CLINIC PROTOCOL

2 times at 4 week intervals.

- 1. Foamer 15** - Use to cleanse the face.
- 2. TCA** - Pour 1.5ml of TCA into a cup and apply to the face using 1 cotton bud. Use TCA 30 in combination with a lower concentration of Cosmo Peel (15 to 20%), for a targeted application to problem areas: visible pigment spots and deep wrinkles. In some cases, you may apply TCA 30 to the full face after careful assessment and testing of the patient's skin tolerance. Do not rinse.
- 3. Panthenol Ceutic** - Massage into the face. Rinse between 4 and 8 hours later.

WHY CHOOSE TCA 30 ?

- + Controlled penetration thanks to buffered TCA
- + Wide range of concentrations:
 - + Epidermal to dermal stimulation for all phototypes
 - + Targeted to full face application

PEEL INTENSITY



MAY BE COMBINED WITH
COSMO PEEL (15 TO 20%)



Stand alone includes: TCA 30 20 ml and cotton buds – For 12 applications

CE 0051 limited to TCA 30 bottle

TCA 30 is a class IIa medical device

Medium peel

COSMO PEEL FORTE

CE 0051

Intended for the treatment of acne scars, senile lentiginos and actinic keratosis.



Kit includes: Foamer 15 40 ml, Cosmo Peel Forte 20 ml, Cosmo Cream 40ml, cup and cotton buds - For 12 applications

CE 0051 limited to Cosmo Peel Forte bottle

Cosmo Peel Forte is a class IIa medical device

PEEL INTENSITY



BENEFITS FOR THE SKIN*

- + Promotes the removal of **acne scars** through combined ingredients
- + Eliminates pronounced **pigmented lesions**
- + Eliminates **actinic keratosis** lesions
- + Boosts **collagen** and **elastin production**
- + Improves appearance of **wrinkles** and **sagging skin**

*Literature based evidences of TCA and phenol peels

ACTIVE INGREDIENTS

Cosmo Peel Forte: Phenol, TCA, glycolic acid
pH ≈ 0.5

Cosmo Cream: Dextran sulfate, allantoin, hyaluronic acid, vitamin C, calendula extract
pH ≈ 4.8

Foamer 15: Glycolic acid, enoxolone
pH ≈ 4.1

POSSIBLE SIDE EFFECTS

Desquamation, erythema, frosting, oedema, redness, pigment rebound, allergy and/or burning sensation may appear. If these symptoms persist for longer than 72 hours arrange a consultation with the patient.

CLINICALLY TESTED



IN-CLINIC PROTOCOL

2 times at 8 week intervals.

- 1. Foamer 15** - Use to cleanse the face.
- 2. Cosmo Peel Forte** - Apply to the face. Wait 1 to 2 minutes. Reapply additional layer if necessary. Do not rinse.
- 3. Cosmo Cream** - Apply to the face, rinse between 4 and 8 hours later.

WHY CHOOSE COSMO PEEL FORTE ?

- + Outstanding results with 1 peel only
- + Controlled and painless application
- + Superior formula (phenol, TCA & glycolic acid) for optimal product efficacy
- + Visibly improved and younger-looking skin

*In vivo efficacy study on 63 patients. European independent contract research organization.

PROVEN RESULTS

94% of patients have **less wrinkles** after 1 Cosmo Peel Forte.*

SYNERGY OF 3 ACTIVES

Phenol to regenerate the dermal architecture and control pain.

TCA to coagulate skin proteins, and accelerate skin renewal and phenol penetration.

Glycolic acid to increase the penetration of TCA and phenol.

EXPERT TESTIMONIAL

“ This peel is a true innovation! Its unique formula ensures a visible improvement of the skin texture (medium peel depth) thanks to a controlled penetration. Plus, it offers a real comfort for patients. ”

Dr. Jean-Luc Vigneron

DERMACEUTIC

LABORATOIRE

「 Skintelligent Science 」

Sole Distributor:

REDO
MARKETING
Your Aesthetic Business Solutions Provider

22A, Jalan 22/5, Seksyen 22, Shah Alam,
40300 Selangor Darul Ehsan.

Tel : +603-5103 6651

Email : sales@redo.com.my

Website : www.redo.com.my