

NOVUMA®

Collagen Stimulating
Dermal Filler

CaHA

REVEAL YOUR

Magic


BURGEON





Advanced Ceramic Technology of Novuma

Smooth Surface

The smooth, round surface of CaHA microspheres reduces friction, lowering granuloma risk and ensuring natural, aesthetic skin compatibility.

Optimized Particle Size

To avoid phagocytosis, particles must be 25-45 μm . Smaller ones ($<25 \mu\text{m}$) are quickly cleared. Novuma ensures this optimal size for efficacy.

High-Density Crystal Structures

Novuma's CaHA microspheres, with large crystal structures, enhance shaping, longevity, and effectiveness.

Unique Composition

Novuma's unique CaHA composition ensures long-lasting, natural rejuvenation with optimal efficacy and reliability.

Product Overview



Safety

100% bioactive ingredient with CE certification



Durability

Sustained effect for 12-24 months



Gel Structure

Easy to inject



Packaging

1 x 1.5cc



Effectiveness

Instant filling, rejuvenation, lifting, contouring, and collagen boost



Content

56-62% (w/w) CaHA microspheres (25-45 μm) in a CMC gel carrier

Powerful Lifting and Correction Effect

Provides instant strong lifting and correction thanks to its firming effect.

Easy Application

Effortlessly applied with a low extrusion force of less than 5 Newtons.

Biocompatible

A biocompatible and biodegradable filler.

Advanced Technology Content

Contains Calcium Hydroxyapatite suspended in a Carboxymethyl Cellulose Gel carrier.

High Production Standards

Manufactured in accordance with ISO Class 7 standards.

Intense Shaping Effect

Offers intense and defined shaping effect due to the high density of microspheres in the carrier gel.

Areas of Application



Cheeks



Marionette Lines



Jawline



Jaw



Nasolabial Folds



Hands

Indication	Average volume of undiluted CaHA	Dilution Ratio	Injection Plane / Technique
Pan-facial rejuvenation	1.5 mL / Side	Most commonly 1:1; up to 1:3	Subdermal, Retrograde linear fanning
Neck	1.5mL	1:2 - 1:4	Immediate subdermal Retrograde linear threads
Decolletage	1.5mL	1:2 - 1:4*	Immediate subdermal Retrograde linear threads
Mild laxity of the upper arm	3 mL / arm	1:2*	Immediate subdermal Retrograde linear fanning
Abdomen	1.5mL / 100cm ²	1:1	Subdermal Cross-hatching or fanning
Buttocks Gluteal sagging, mild dermal irregularities	1.5mL per side	1:2 & 1:6, layered	Deep dermal cross-hatching
Mild laxity of the legs	1.5mL / 100cm ²	1:2*	Immediate subdermal horizontal "rasping"
Cellulite	1.5mL per side	1:1	Subdermal vectored fanning
Striae	1.5 - 3 mL per session	1:1	Subcutaneous to superficial dermis Microbolus or retrograde linear thread

*Recommended Treatment Paradigm for Diluted & Hyperdiluted CaHA

Treatment *Application*



Midface



Face



Chin Line



Perioral

Treatment *Results*



*1 month after injection

novuma®

Sole Distributor:

REDO
MARKETING
Your Aesthetic Business Solutions Provider

22A, Jalan 22/5, Seksyen 22, Shah Alam,
40300 Selangor Darul Ehsan.

Tel : +603-5103 6651

Email : sales@redo.com.my

Website : www.redo.com.my